

Product Catalog Clinical Range 2021



Experience The Refinement

Shiva Products

ISO 13485 : 2016 (TUV SUD Certified)

Proudly Exporting to 40+ Countries Worldwide

INDEX

NEW LAUNCH

Soft Liner.....	05
HF Gel.....	06
Blitz White.....	07
Articulating Paper (Horseshoe).....	08
CeraSeal-B.....	09
MTA.....	10
Twin Kleen.....	11
Easy Tray.....	12
Hemostaz Solution / Gel.....	13

ENDO RANGE

Sealmax-R.....	14
Calx.....	14
Calx-O.....	15
Calahyd-RC.....	15
Endo Paste.....	16
Gsol.....	16
Chloro-HX.....	17
Sodium Hypochlorite.....	17
F-Sol.....	18
D-Vitz.....	18
Endo Gel & Endo-L.....	19
T-Fill.....	19

PREVENTIVE RANGE

Find Fast.....	20
Zirprophy.....	20
Pro-Paste.....	21
Plaque-D.....	21

PROSTHO RANGE

Mock-up wax.....	22
Bite Registration Wax.....	22
Green Tracing Sticks.....	23
Shellac Base Plates.....	23
Impression Compound.....	24
Special Tray Plates.....	24
Modelling Wax.....	25
Casting Wax Sheets.....	25
Inlay Sticks.....	26
Articulating Paper.....	26
Die Stone Plaster (Pink).....	27
Reinforcement Mesh.....	27
Beading Wax.....	28
Boxing Wax.....	28
Cold Cure Powder / Liquid.....	29
Cold Mould Seal.....	29
Heat Cure Powder / Liquid.....	30
Sticky Wax Sticks.....	30

RESTORATIVE RANGE

ETC-37.....	31
Mercury.....	31
Calcium Hydroxide Powder.....	31
ZIEGO (Zinc Oxide + Eugenol Kit)	32

OTHER

Lubri Spray.....	33
Arch Bar.....	33
Glass Beads.....	34
Relief Wax for Ortho.....	34
Angular Alcohol / Spirit Lamp.....	35
Carding Wax / Teeth Wax.....	35



COMPANY PROFILE

An ISO 13485 : 2016 accredited company catering a wide range of products such as dental material, dental products, dental alloy, jewellery wax, etc...

An Overview

Shiva Products, established in the year 1998, an eminent company is a manufacturer for Dental Products and other specialty waxes. With more than two decades of experience in this industry, we are proud to be recognized as a renowned manufacturer and exporter of premium quality Dental Goods, Jewellery Waxes, Investment Casting Waxes, Modelling Waxes, Dental Alloys, Dental Instruments, etc., which are specifically made according to international quality parameters. We are a team of experienced professionals working with a common goal of quality production, thereby, winning the trust and faith of thousands of national and international clients.

The Company has ventured into the field of dentistry since 1998 with 2 products in hand. With Team work & focus on **RRR strategy (Reliable Responsible & Reasonable)**, we currently have more than 80 Products catalogued and many in pipe line. Company has become one of the leading manufacturer in Domestic & well-known in International Market. The Strong relationship and the trust of consumers, trade and the dental profession built over two decade of operations in India has made 'MAARC' a trusted Brand.

'MAARC' products have the reputation of "**RIGHT QUALITY RIGHT PRICE**" in India as well as International market. Our ISO 13485:2016 manufacturing standards helps us to deliver quality dentistry products at affordable rates to masses. Our goal is to build confident relationship with our customers.

We also cater to manufacturers in neutral packing and contract manufacturing can also be done.

DENTAL SEGMENT:

Clientele:

01. Domestic: We are currently selling to approx. 1000-1200 Distributor / Dealers (many states of India), approx. 500-600 Laboratories within Mumbai, 4000-5000 dentists are been catered directly / through distributor/dealer network through our company within Mumbai + Institutional supply + government Tender supply.

02. Exports - Currently supplying to approx. 40 countries across various continents

Certifications: Shiva Products has several certifications as under.

- a) EN ISO 13485 : 2016
- b) Class I device products are all CE certified
- c) EAR Obelis SA -Belgium
- d) Registration with various Chamber of commerce in Mumbai for smooth functioning of export orders.

COMPANY PROFILE

Vision

We are led by global vision and drive to manufacture various dental material comparable to international standards so as to offer optimal customer satisfaction.

With a goal to establish strong relationships with more and more customers, we offer our Products at extremely affordable price. It is also a continuous effort on our part to deliver the orders within the stipulated time.

Assured Quality

We take utmost care of quality while producing our goods, which meet international quality norms in all respects. Shiva Products has been certified with an ISO 13485 : 2016 certification, which is a testimony in itself to our quality production. Each operation is executed under the strict supervision of trained professionals in accordance with standardized production norms and methods, thus leaving no stone unturned in ensuring premium quality.

Infrastructure

Shiva Products has a state-of-the-art manufacturing unit, where latest manufacturing machines and equipment contribute substantially in manufacturing global quality dental products. Our manpower includes engineers, dentists, technicians, designers, R&D team, QC, marketing professionals, quality personnel in addition to some semi-skilled workers. We work as a team to achieve our vision.

In-process Quality Analysis: Our manufacturing unit has its own lab and checking facilities wherein quality analysis is been done on daily basis for RM, in-process and final product before its sent to the packing zone.

The most advantageous and beneficial technique of production supplied by our R&D team give us an edge over our competitors. Our warehouse is a boon, as it facilitates safe storage of bulk products to meet all types of market demands.


Future Plans

Our Company want to take advantage of the booming export market to add to its market share and not lose the opportunity to get an insight into the behavior of western markets and its consumers. Led by a global vision and a drive to maintain quality upto international standards, our company strives to produce products conforming to international standards.

We would like more distributors to ADD their companies to our network and build beautiful Smiles all over!!

New Launch

Soft-Liner

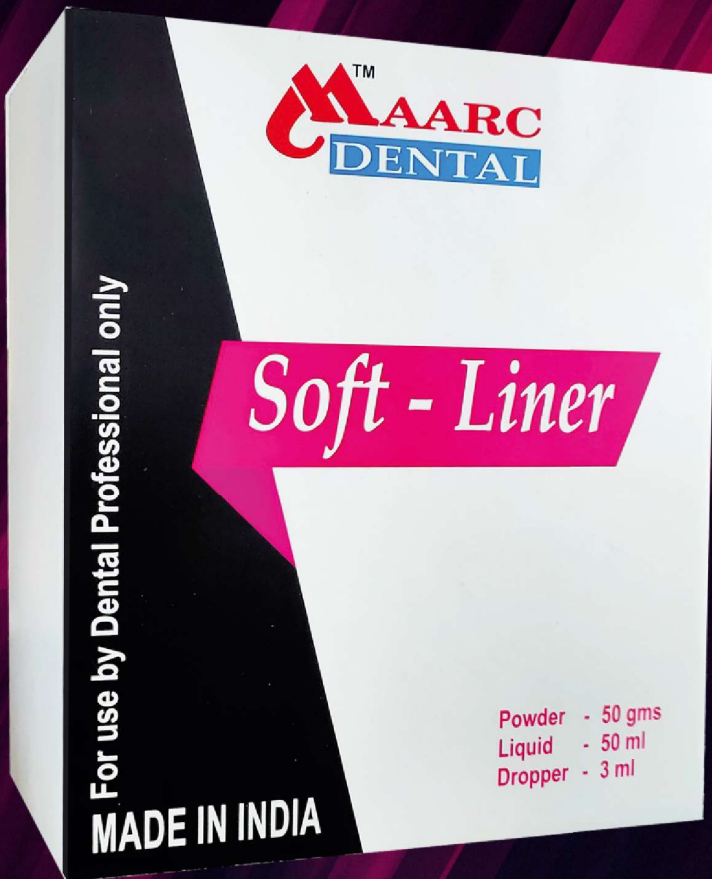

Packaging	Product Code
50 gms Powder 50 ml Liquid 3 ml Dropper	8005/001

- Comfortable Tissue Conditioning
- Absorbs Shock Efficiently
- Enhances Denture Stability
- Excellent Denture Adhesion

Shelf life and storage : Best before 24 months under proper storage.

New Launch

HF Gel



10% Buffered Hydrofluoric Acid Gel


- Better Stability
- Fast Porcelain Etching
- Controlled Flow
- Precise Placement
- Selective Etching
- High Visibility

Packaging	Product Code
1 ml x 2 syr. 4 applicator tips	9801/1X2

Shelf life and storage : Best before 24 months under proper storage.

New Launch

Blitz White

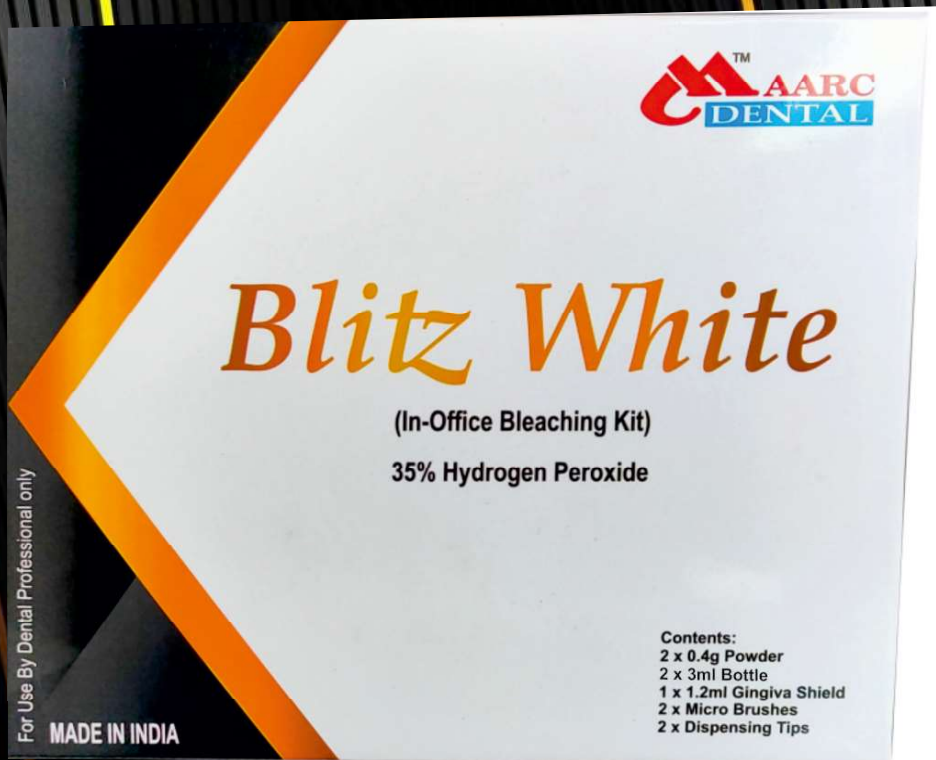


- In-office system with De-sensitising agent
- Less post procedural sensitivity
- Quicker Whitening Result
- Contains Fluoride for Re-mineralization

Packaging	Product Code
2 x 0.4g Powder 2 x 3 ml Bottle 1 x 3gm Gingiva shield 2 x Micro Brushes 2 x Dispensing Tips	8801 / 001

Shelf life and storage : Best before 24 months under proper storage.

New Launch

Articulating Paper (Horseshoe Shape)

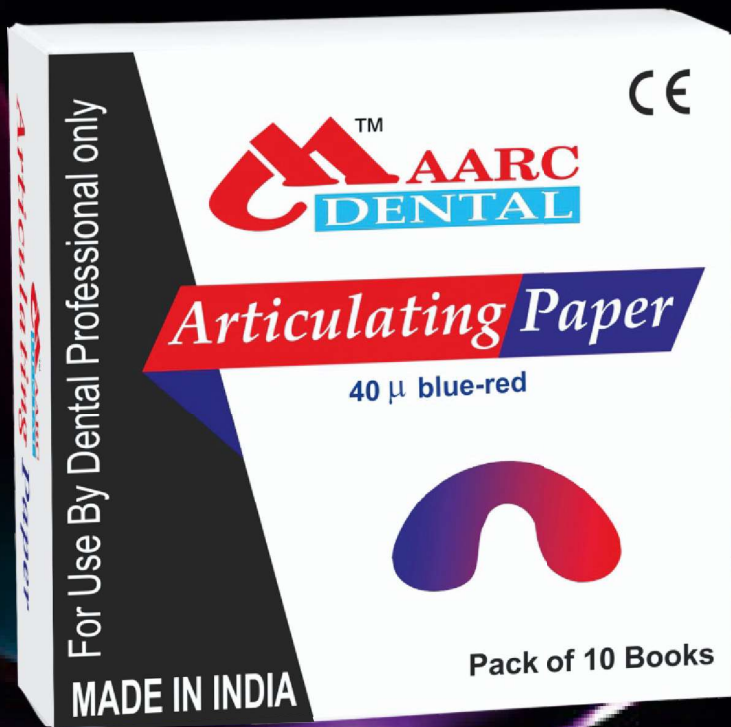

Packaging	Product Code
Pack of 10 books	5523/001

40μ Horseshoe, Blue-Red

New Launch

New Launch

CeraSeal-B

Packing	Product Code
1 gm powder, 5 ml liquid, measuring spoon	8501/001



- Excellent Handling
- Hermetic seal
- Radiopaque
- Suitable as lining material
- Pure Mineral Tech :No Discoloration
- Bactericidal
- Biocompatible
- Good radiopacity

Shelf life and storage : Best before 24 months under proper storage.

New Launch

MTA

Packing	Product Code
1 gm powder, 5 ml liquid, measuring spoon	8601/001



- Excellent Handling Properties
- Fine Grain
- Fast Setting
- Pure Mineral Tech : No Discoloration
- Bactericidal effect due to High pH
- Biocompatible
- Promotes mineralization of tissue
- High Strength make is suitable as base restoration material
- Good radiopacity

Shelf life and storage : Best before 24 months under proper storage.

New Launch


Twin Kleen

Packing	Product Code
100 Capsules	3014/100

The All in One Irrigating Solution

Benefits of using Maarc TWIN KLEEN

- Single Step Irrigation
- Antibacterial Solution
- Mild Chelating Agent
- Proteolytic Agent
- Lubricating Agent
- Prevents Smear Layer
- Marked Reduction in Accumulated Hard Tissue Debris
- Less Post Operative Pain
- Conditioning of Dentin to Receive Sealer
- Reduces the Inventory of Solution

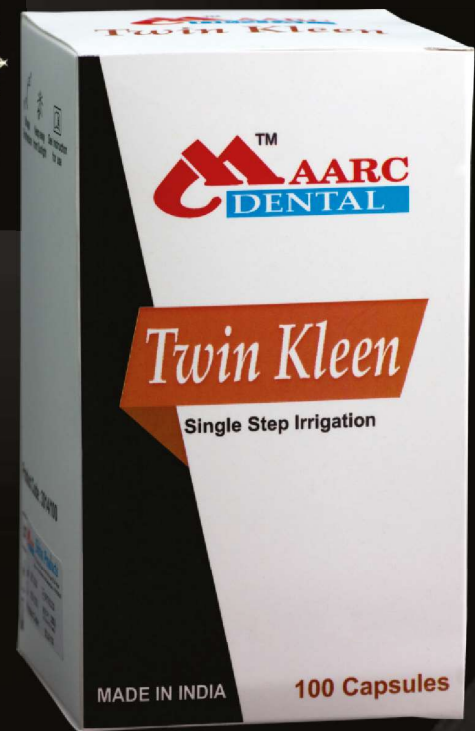

Just Mix Powder with Sodium Hypo

Shelf life and storage : Best before 24 months under proper storage.

Eazy Tray

Packing	Product Code
100 gms	9701/100

New Launch

- For Lucia Jig
- Bite Stop
- Special Tray



Shelf life and storage : Best before 24 months under proper storage.

Hemostaz Solution / Gel

New Launch

Packing	Product Code
15ml	3016/015
3gm Syringe with 10 Pcs Applicator tips	3018/003

- Stop Bleeding in Seconds, and Decreases the Need to retake Impression
- Hemostatic Agent for Crown & Bridge Procedures
- Viscous - Stay in Place when Delivered
- Unique Binding & Coating Agents



Shelf life and storage : Best before 24 months under proper storage.

ENDO RANGE

Sealmax – R



Known as : Resin based root canal sealer

Packing	Product Code
10 gm powder, 10 ml liquid, dispensing scoop - 1N	8001/001

Description:

▶▶ Sealmax-R contains fine particles to ensure proper sealing

Shelf life and storage: 24 months under proper storage. Always close the cap immediately after use.

Properties & Advantages

- ▶▶ Eugenol Free
- ▶▶ Insoluble in water and non degradable
- ▶▶ Minimum shrinkage at room temperature

Benefits :

- ▶▶ Grain Free
- ▶▶ Hydrophobic
- ▶▶ Bacteriostatic
- ▶▶ Negligible Shrinkage
- ▶▶ Complete Impervious Seal
- ▶▶ Excellent Adhesion
- ▶▶ Tissue Tolerant
- ▶▶ Most Ideal Viscosity
- ▶▶ Best Intracanal Sealing
- ▶▶ Excellent Radio Opacity

Endo Range

Calx

Known as : Calcium Hydroxide Paste (Regular)

Packing	Product Code
1.5 x 2 Syringe + 3 Pcs (18 G) Applicator tips	9104 /1.5x2

Description :

▶▶ It is premixed high pH Calcium Hydroxide Water based paste with stabilizer

Indication :

▶▶ Flowing Canal ▶▶ Hard Tissue Formation ▶▶ Root Resorption ▶▶ Radiolucent

Properties & Advantages

- ▶▶ Ready to use, radiolucent, premixed, non separating Calcium Hydroxide Paste which is useful in periapical infection
- ▶▶ Used as a liner.
- ▶▶ It reduces the hypersensitivity of the dentine and helps the formation of secondary dentine.
- ▶▶ Non Separating Homogenous Mix
- ▶▶ Ideal Viscosity & Micronised Calcium Hydroxide for Effective Application
- ▶▶ Easy Insertion in Canal Thanks to "MAARC ADHERE TECH"

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.



ENDO RANGE



Calx-O



Known as : Water Based Calcium Hydroxide Paste (With Barium Sulphate)

Packing	Product Code
1.5 gm x 2 Syringe with 3 Applicator tips	9102/1.5x2
3gm Syringe with 3 Applicator tips	9105/003
For Sale In India Only	

Indications :

- ▶▶ Flowing Canal
- ▶▶ Apexification - Hard tissue barriers
- ▶▶ Root Resorption

Advantages : Radio-opaque with Excellent bio-compatibility with no toxic effect on cells. Easy handling and easy to apply. Ideal for infected root canal and vital pulpotomies in deciduous teeth. It can also be used in conjunctions with gutta percha points and regular root canal sealants.

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.

Endo Range

Calahyd-RC



Known as : Calcium Hydroxide Paste With Iodoform

Calahyd - RC is premixed Calcium Hydroxide Paste containing Iodoform. The addition of iodoform to calcium hydroxide improves radiopacity and adds an antibacterial agent to the paste. Non hardening paste, solublized calcium hydroxide remains active in root canal for a longer time.

Benefits :

- ▶▶ Non-Separating Homogenous Mix
- ▶▶ Bacteriostatic & Antibacterial Properties
- ▶▶ Effective in Abscess
- ▶▶ Promotes Apexification
- ▶▶ Easy Retrieval from the Canal Using ENDO-L (EDTA LIQUID)+10% Diluted Citric Acid Solution

Applications : For pulp capping and pulpotomy. Apexifications and hard tissue formation. Temporary sealing material for infected root canals.

Method of Use : After root canal preparation, clean and dry root canal. Attach disposable needle to the Syringe and check the flow of paste through the needle. Insert needle into the canal, about 2mm short of the apex. Fill canal completely by pushing the plunger while slowly withdrawing the syringe. Clean excess paste with a sterile cotton pellet.

Packing	Product Code
1.5 x 2 Syringes with 4 Applicator tips	9103/15X2

Storage & Shelf Life : Best before 24 months keep away from direct flame. Store in a cool and dry environment. Always close the cap immediately after use. Do not freeze.

ENDO RANGE

Endo Paste



Benefits :

- ▶▶ **GLIDE YOUR FILES:** With Unmatched Lubrication
- ▶▶ **EASYCOAT:** Coats File With Ease
- ▶▶ **REMOVE SMEAR LAYER EFFECTIVELY**
- ▶▶ **EFFERVESCENCE:** Reacts With Sodium Hypo To Flush Debris
- ▶▶ **HYDROPHILIC:** Easy Wash Off
- ▶▶ **NO WASTAGE:** Better Stability Non Drying & No Phase Separation

Description

It is water soluble Ethylene Diamine Tetra Acid (EDTA) Paste for root canal cleaning

Packing	Product Code
3gm Syringe with 3 Applicator tips	9203/003

Indication of Use

- ▶▶ Chelating (Calcium Binding) agent
- ▶▶ For chemical and mechanical cleaning of the root canal
- ▶▶ Shaping of root canal
- ▶▶ To be used with sodium hypochlorite. Don't use on soft tissues light skin.

Advantage :

Contain 3 lubricant for easy reaming root canal

Shelf Life : 24 months from date of manufacturing. Always close the cap immediately after use.



Known as : Gutta Percha Solvent D-limonene based G. P solvent for removing Gutta Percha at the time of root canal re-treatment

Packing	Product Code
15ml	3006/015

Indications:

- ▶▶ To remove all types of root canal filling materials including Zinc-Oxide-Eugenol cement
- ▶▶ Fitting of the primary gutta percha point in root canal filling techniques using the lateral condensation method.
- ▶▶ Retreatment after root canal filling by gutta percha condensation.
- ▶▶ For removing gutta percha at the time of root canal re-treatment

Benefits :

- ▶▶ G-Sol instantly softens GP (**Rapid Action**)
- ▶▶ Easy removal of Sealer once GP soften
- ▶▶ Orange oil based (**Pleasant flavour**)

Product Benefits :

Non hazardous/non carcinogenic unlike chloroform/trichloroethylene	Less volatile as compared to chloroform / trichloroethylene
Based on natural substances d-limonene, doesn't cause irritation to oral mucosa and periapical tissue	Evaporation of solvent is less due to low volatility

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.

Gsol



ENDO RANGE



Chloro-HX - 2%



2% Chlorhexidine Solution that is used to clean & disinfect before many dental procedures. This greatly reduces the potential for post-operative sensitivity caused by residual Bacteria.

Packing	Product Code
100ml	3010/100

Shelf Life : 24 months from the date of Manufacturing.

Endo Range


Sodium Hypochlorite

Packing	Product Code
(3%) 100ml	3009/100
(5%) 100ml	3004/100
(3%) 250ml	3009/250
(5%) 250ml	3004/250

Properties & advantages :

- ▶▶ Used as endodontic irrigant, debridement and disinfection of root canals during and after instrumentation

Shelf life and storage: 24 months under proper storage.

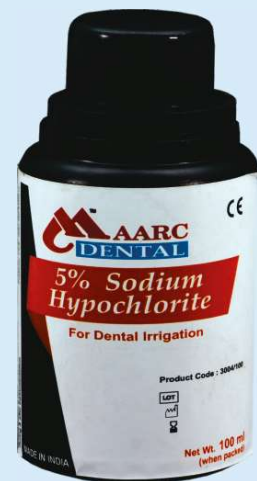

ENDO RANGE



F-sol



Known as : Formocresol Solution

Packing	Product Code
15 ml	3012/015

INDICATIONS

Indicated as a disinfectant and devitalizing solution used as part of endodontic treatment.

STORAGE

Store between 59°-86°F (15°-30°C). Protect from freezing.

Shelf life and storage : Best before 24 months under proper storage.

D-Vitz (Pulp Devitalizer)



It is a pulp devitalizing Paste which provides powerful pulp devitalizing properties because of paraformaldehyde & lidocaine. It is Arsenic free paste

Packing	Product Code
3 gms syringe + 5 Applicator tips	9003/003

Shelf life and storage: 24 months under proper storage.

Endo Range

ENDO RANGE



Endogel-RC

Known as : Endogel-RC / Endo-L

Packing	Product Code
3 gms syringe with 3 Applicator tips	9202/003
100 ml.	9201/100

Properties & Advantages :

- ▶▶ Effective chelating, lubrication, deburring water soluble material.
- ▶▶ It facilitates root canal preparation.

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.

Benefits :

- ▶▶ 17% Liquid based EDTA
- ▶▶ Used to flush out the debris from canal
- ▶▶ Removes the smear layer from dentinal walls
- ▶▶ Also helps in removal of calcium hydroxide oil based from the canals
- ▶▶ Mechanically Activated using a endo brush

Endo-L



Product Description :

T-Fill is ready to use temporary filling material self-curing under humidity for temporary filling of cavities.

Packing	Product Code
40gm pack, mint flavour, colour white	5507/040

Indication of use :

- ▶▶ For use as Temporary filling material after Endodontic treatment
- ▶▶ For use as Temporary Inlay material

Properties :

- ▶▶ Eugenol Free offer non irritating properties.
- ▶▶ Higher adhesion to teeth structure.
- ▶▶ Micronised and homogeneously mixed for grain free consistency. Gives best adaptability.
- ▶▶ Quick setting in moist conditions
- ▶▶ There is minor but even expansion on setting so no cracks develop, giving best sealing hence Protecting to Pulp.

Storage : After opening store in refrigerator (don't Freeze) .

Shelf life and storage : 24 months under proper storage.

T-Fill




Benefits :

- ▶▶ Excellent Adaptability
- ▶▶ Non Sticky to Instrument
- ▶▶ No Cracks: Even Expansion Due to Even Grain Size
- ▶▶ Eugenol Free
- ▶▶ Pleasant Flavor : Mint
- ▶▶ Neutral Taste
- ▶▶ Minimum Moisture Contamination: Dual Cap For Air Sealing

PREVENTIVE RANGE

Find Fast

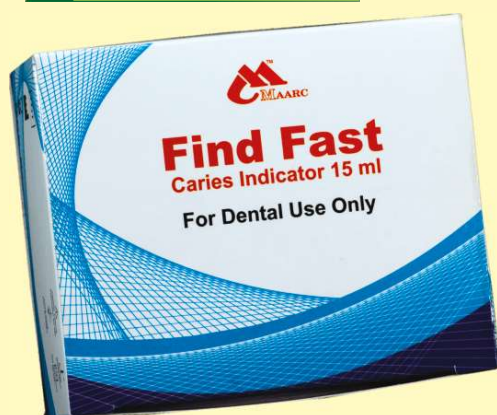

Known as : Caries Indicator

Packing	Product Code
15ml	3005/015



Properties & Advantages :

- ▶▶ Find Fast is made from special dye
- ▶▶ It stains protein ensures uninfected areas is not stained
- ▶▶ Identifies carious dentin so you can ensure the maximum amount of inner dentin is preserved and infected dentin is removed
- ▶▶ Identifies carious dentin ensures the maximum amount of inner dentin is preserved and infected dentin is removed.
- ▶▶ Easily locate root canal orifices and calcified canals.

Shelf life & storage: 24 months under proper storage.

Zirprophy

Known as : Prophylaxis Paste

Contains : Zirconium Silicate.

Packing	Product Code
45 gms	9001/045
340 gms	9001/340

Properties & Advantages :

- ▶▶ Zirprophy contains Zirconium Silicate.
- ▶▶ It does not break down as pumice from coarse to fine but rather removes stain smoothly due to fine particles.
- ▶▶ Zirprophy is to be used light handedly to avoid damage to enamel
- ▶▶ Thixotropic Oil Free
- ▶▶ Splatter Free
- ▶▶ Fluoride Free
- ▶▶ Spearmint Flavor

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use. Do not freeze.



Benefits :

- ▶▶ Zirconium silicate abrasive media
- ▶▶ Easy removal of smoking and tar tar stains
- ▶▶ Controlled particle size (Minimum post scaling sensitivity)
- ▶▶ Better polish
- ▶▶ Fluoride Free

PREVENTIVE RANGE

Pro-Paste



Known as : Prophylaxis Paste

Packing	Product Code
90 gms	9002/090

Description:

▶▶ Pro-Paste is a pumice paste for traditional polishing and stain removing use

Properties & Advantages :

- ▶▶ Thixotropic Oil Free
- ▶▶ Splatter Free
- ▶▶ Mint flavor
- ▶▶ Do not Freeze

Shelf life and storage : 24 months under proper storage.
Always close the cap immediately after use. Do not freeze.




Plaque-D

Packing	Product Code
2gm x 4 syr. + 4 applicator tips	9301/010

PLAQUE-D : Plaque Disclosing Gel is made of special dye which stains plaque. As a result, one gets clear idea of brushing irregularities. Dentist can easily make patients understand in which area they have to focus more and which technique of brushing to be use for more effective cleaning. This is one of the best way to educate patients about their brushing technique and teach them proper brushing.

Dentist can give one syringe to patient to try the same at their home.


Shelf life and storage : 24 months under proper storage.

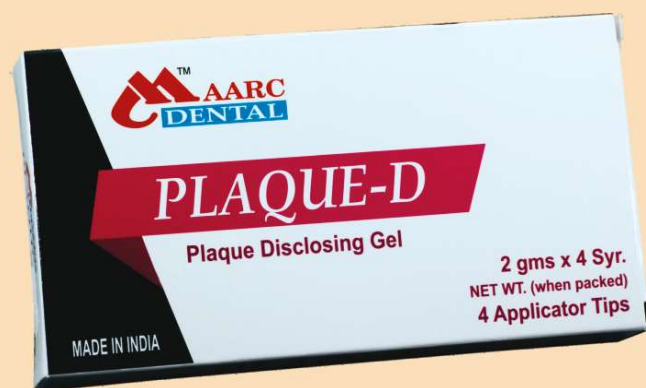

PROSTHO RANGE



Mock-Up Wax CE

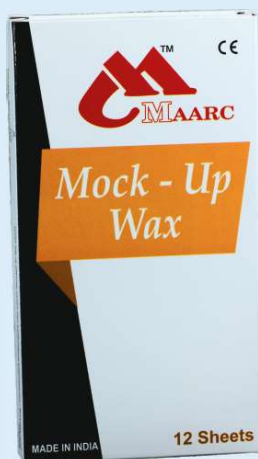

Packing	Product Code
2 Sheet	2201/225
200 gms	2201/200

Color : White

Product features :

- ▶▶ It simulates the final outcome. The result is temporary and can be changed according to the patients desire
- ▶▶ It can be repeated as many times as needed in order to select the best possible solution
- ▶▶ It helps create trust between the patient and the dentist
- ▶▶ It allows the dentist to evaluate the patient's desires and change the treatment plan accordingly

Shelf life and storage : Best before 60 months under proper storage.

Thickness : 1.3mm - 1.5mm. (Custom Size / Thickness Catered),

Size : 160mm X 90 mm

Packing : Each Packet of 'MAARC' Mockup Wax contains 12 sheets wrapped in non adhesive Silicone Paper to Facilitate easy removal.

Prosthodontics Range

Thickness : 1.7mm to 2.0mm (Custom Thickness Catered),

Size : 160mm X 90 mm

Packing : Each Packet of 'MAARC' Bite Registration Wax contains 10 sheets wrapped in non adhesive Silicone Paper to facilitate easy removal. Available in 250gms pack appx. Bite Wafers pack of 10

Available in : 3 variants - Hard, Extra Hard & Zero Tolerance

Packing	Product Code
Bite Registration Wax (Aluminium Filled)	2202/010
Classic BRW Extra Hard (Aluminium Filled)	2207/010
Perfect Bite (Zero tolerance)	2213/010
Bite Wafers	2210/010

Properties & Advantages :

- ▶▶ Contains Aluminium particles which gives even heating
- ▶▶ Dimensionally stable, does not shrink or distort.
- ▶▶ Ideal for checking and maintaining accurate occlusal records.
- ▶▶ When heated at 55°C it turns in to 'Dead Soft' in less than 1 minute.
- ▶▶ Adapts to every details and sets by water or room temperature i.e. 27°C.

Shelf life and storage : Best before 60 months under proper storage.

Bite Registration Wax CE



Benefits :

- ▶▶ **Accurate Bite:** Even Heating Due to Aluminum Particles.
- ▶▶ **No Shrinkage:** Negligible Shrinkage Due to Crystallization Around Aluminum & Resin.
- ▶▶ **No Distortion:** Very Tough so Lesser Change of Distortion.

PROSTHO RANGE

Green Tracing Sticks CE



Thickness : 113 x 9 mm dia weighing appx. 10gms.

Packing	Product Code
10 Sticks	1501/010
15 Sticks	1501/015
100 Sticks	1501/100

Indication :

- ▶▶ Low fusing material primarily used for border extensions on impression trays.
- ▶▶ High viscosity also makes it ideal when recording the full depth of the sulcus.
- ▶▶ Modify dentate tray by building up material where tooth has been extracted.
- ▶▶ Make functional and relining impressions.

Properties & Advantages :

- ▶▶ Suitable for muscle trimming work.
- ▶▶ Fast and uniform plasticity.
- ▶▶ Quick hardening in cold water.
- ▶▶ Wide working area.

Shelf life and storage :

Best before 60 months under proper storage.

Shellac Base Plates CE

Thickness : Upper / Lower Base Plate: 1.3mm, 1.7mm, 2.0mm, 2.5mm and 3.0mm (+/- 0.15mm)
(Custom Thickness Catered)

Colour : Brown / Pink / Aluminium

Packing	Product Code
Brown Base Plate (8 Upper + 4 Lower)	1001/084
Yellow base Plate (8 Upper + 4 Lower)	1024/084

Type : It comes in different formulations for different climatic conditions (LMP / HMP / AI. Filled)

Properties & Advantage

- ▶▶ They are ready to use Maxillary & Mandibular plates - 8 Upper & 4 Lower.
- ▶▶ Optimum heat resistance to minimize overheating of the Plate.
- ▶▶ Easily adaptable on heating and easy to work with.
- ▶▶ Retain the shape stability and do not warp back on cooling.
- ▶▶ Optimum mechanical strength to resist breaking during Try-in denture
- ▶▶ Does not adhere to plaster.



Shelf life and storage : Best before 24 months. Store below 30°C under proper storage and avoid from direct sunlight.

PROSTHO RANGE

Impression Compound CE



Packing	Product Code
Pack of 5 cakes of appx. 200gms	1503/005

Properties & Advantages :

- ▶▶ It softens at 60°C.
- ▶▶ It is excellent for taking primary impression of edentulous teeth and becomes stable after cooling and are shrinkage free.
- ▶▶ It can be used with matrix band for single tooth impressions.

Shelf life and storage : Best before 60 months under proper storage.

Prosthodontics Range

Special Tray Plates CE

Packing	Product Code
10 pcs.	1309/010

Benefits Of Special Tray Plates :

- ▶▶ Time Saving and economical as compared to light cure and cold cure tray.
- ▶▶ No toxic gases as in cold cure.
- ▶▶ STP is low fusing thermo-plastic compound with high flow. So does not melt wax spacer and flows through stoppers.
- ▶▶ It has heat conducting properties, so no worries when applying hot border moulding compound. Heat spreads immediately and plate dimension remains unchanged.
- ▶▶ Stronger than conventional Shellac Plates.
- ▶▶ Easily fused to each other so handle can be made from excess.
- ▶▶ Excess can be cut easily with scissors when hot and when cold, trim with burs, abrasive wheels, etc.
- ▶▶ Borders can be folded easily.
- ▶▶ Modify the tray anytime so best border moulding can be achieved.

Shelf life and storage : Best before 24 months under proper storage.



PROSTHO RANGE

Modelling Wax / Sheet Wax / Base Plate Wax / Hard Wax CE



Thickness : 1.3mm - 1.5mm. (Custom Size / Thickness Catered),

Size : 160mm X 90 mm

Packing	Product Code
Modelling Wax Regular	(12 Sheet-2001/012)-(500 gms-2001/500)
Modelling Wax Red	(12 Sheet-2002/012)-(500 gms-2002/500)
Plasti Wax Type 6	(12 Sheet-2015/012)
HARD WAX	(12 Sheet-2010/012)-(500 gms-2010/500)

Color : It comes in various shades such as pink, light pink, red.

Type : Soft, Medium, Hard & Plasti Wax.

Properties & Advantages :

- ▶▶ The wax is formable at slight increased temperature.
- ▶▶ Sufficient hard and resistant on cooling to prevent occlusion changes during trying with patient before making Final Denture to prevent failure of final denture.
- ▶▶ Easy & Clean carve on teeth.
- ▶▶ Bends without cracking.

Shelf life and storage :

Best before 60 months under proper storage.

Casting Wax Sheets CE

Thickness : 0.3 to 0.6 mm

Size : 160mm x 90 mm

Colour : 'Maarc' Casting Wax sheets are thin wax sheets. Available in Green color for cobalt chrome cast partial dentures.

Packing : Available in 200gms pack. The Sheets are wrapped in non adhesive Silicone Paper to Facilitate easy removal.

Properties & Advantages :

- ▶▶ The Sheets are sufficiently thin to have thinner casting as possible without defects to reduce machining.
- ▶▶ They easily burn out during castings.

Shelf life and storage :

Best before 60 months under proper storage.



Packing	Product Code
Packing 200 gms	2206/200

PROSTHO RANGE



Inlay Sticks CE



Thickness : L 114 x 8 mm dia of Big Sticks

Packing	Product Code
10 Sticks (70 gms)	2403/010

Properties & Advantages :

- ▶▶ It comes in stick form.
- ▶▶ It is used in institutes by students for easy application.

Shelf life and storage :

Best before 60 months under proper storage.

Prosthodontics

Articulating Paper CE

Packing	Product Code
40 μ 1 Box Red	5515/001
40 μ 1 Box Blue	5519/001
40 μ 1 Box Red / Blue	5511/001
100 μ 1 Box Red	5517/001
100 μ 1 Box Blue	5521/001
100 μ 1 Box Red / Blue	5513/001



Properties : All films are available with double-sided color coating for the marking of antagonistic contacts.

Articulating paper is a diagnostic tool used in dentistry to highlight occlusal contacts and the distribution of occlusal forces. That is, it marks those points on the teeth where the teeth contact during biting and grinding. Articulating paper is made of a thin, non-adhesive paper strip covered in fluorescent ink or dye-containing wax. A strip of articulating paper is placed between the teeth while the desired mandibular movements are performed.

Thickness: 40 μ / 100 μ

PROSTHO RANGE



Die Stone Plaster (Pink) CE



For crown and bridge, dies and cast partials. For basing dental arches and opposing models. For making study models. In use where economical yet tough, non brittle and durable models are required.

Packing	Product Code
30 kg	4101/030

Type : IV - Specification :

Water/ Powder	24-26ml/100gm	Hard Set (Initial)	30 Minutes
MixingTime	45 seconds wet	Wet Compressive Strength	400 kg/cm2
WorkingTime	5 minutes dry	Dry Compressive Strength	800 kg/cm2
SettingTime	8-12 minutes	Linear Expansion (Max)	0.08%

Shelf life and storage :

Best before 24 months when stored in cool and dry place.

Reinforcement Mesh CE


Thickness : 0.25 mm

Type : Golden and Silver

Packing	Product Code
Golden Pack of 10 Pcs	5502 / 010
Silver Pack of 10 Pcs	5501 / 010

Properties & Advantages :

- ▶▶ Re-inforcement Mesh is made from 316 grade stainless steel plated with gold or plain polished .
- ▶▶ These Mesh are very much flexible and can easily adapt any shape.
- ▶▶ It is 0.25 mm thin so can be easily layered between acrylic.
- ▶▶ It gives denture desirable strength.

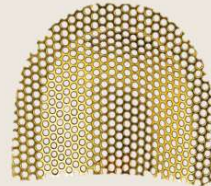

PROSTHO RANGE



Beading Wax CE

Packing	Product Code
Pack of 12 strips	2309/012

Thickness : 6mm x 6mm x 250mm appx.

Packing : It comes in the pack of 12 strips wrapped in non adhesive Silicone Paper to facilitate easy removal.

Properties & Advantages :

- ▶ It is sticky enough to easily adapt at the periphery or edge of alginate impression / final impression by hand pressure.
- ▶ It creates the base of the cast and preserves the outer portion of the cast.

Shelf life and storage : Best before 60 months under proper storage.



Boxing Wax CE

Packing	Product Code
6 Sheets-12"	2204/006
6 sheets-6"	2203/006

Thickness : 160 x 45 x 1.5mm

Packing : Pack of 6 Sheets wrapped in silicon paper to facilitate easy removal.

Properties & Advantages :

- ▶ It is soft and tacky for boxing in impressions.
- ▶ Adheres easily to most of the surfaces.

Shelf life and storage :

Best before 60 months under proper storage.



PROSTHO RANGE

Cold Cure / Rapid Repair Powder & Liquid €



Packing	Product Code
Cold Cure Powder Clear	(100 gms-6201/100)-(400 gms-6201/400)
Cold Cure Powder Pink	(100 gms-6202/100)-(400 gms-6202/400)
Cold Cure Powder Light Veined	(100 gms-6203/100)-(400 gms-6203/400)
Cold Cure Liquid	(100 ml-6301/100)-(400 ml-6301/400)
Cold Cure Universal Pack Clear	400 gms + 400 ml(6504/001)
Cold Cure Universal Pack Pink	400 gms + 400 ml(6505/001)
Cold Cure Universal Pack Light Veined	400 gms + 400 ml(6506/001)



Properties & Advantages :

- ▶▶ This is a 2 part rigid dental acrylic used for making dental appliances.
- ▶▶ Contains both powder and liquid components.
- ▶▶ Pink Dental Acrylic stimulates the fleshy gum tissue of the mouth.
- ▶▶ By combining this with pre-made acrylic teeth, or with your own castings made from tooth shade acrylic can make or repair dentures.
- ▶▶ For best results use with MAARC cold cure liquid.

Description & Features:

- ▶▶ Working time of 4 minutes
- ▶▶ Rapid bench cure time of 14 minutes
- ▶▶ Also useful in orthodontic work

Indication for use : Used for full or partial upper or lower dentures.

Shelf life and storage : Best before 36 months under proper storage.

Cold Mould Seal / Separating Medium €

Packing	Product Code
110 ml	6401/110
5 Liter	6401/005

Properties & Advantages :

- ▶▶ Cold Mould Seal is a specially developed formulation of separating solution for universal use, suitable for both packing / pressing technique & cast moulding.
- ▶▶ Makes tough film with elasticity which is unbreakable in pressure.
- ▶▶ Film does not chip off from plaster at pressure. High quality of the raw material guarantees optimum separation with both techniques.

Shelf life and storage : Best before 36 months under proper storage.



PROSTHO RANGE

Heat Cure / Cross Linked Powder & Liquid CE



Packing	Product Code
Heat Cure Powder Clear	(100 gms-6001/100)-(450 gms-6001/450)
Heat Cure Powder Pink	(100 gms-6002/100)-(450 gms-6002/450)
Heat Cure Powder Light Veined	(100 gms-6003/100)-(450 gms-6003/450)
Heat Cure Liquid	(100 ml-6101/100)-(225 ml-6101/225)
Heat Cure Universal Pack Clear	450 gms + 225 ml + 110 ml CMSL(6501/001)
Heat Cure Universal Pack Pink	450 gms + 225 ml + 110 ml CMSL(6502/001)
Heat Cure Universal Pack Light Veined	450 gms + 225 ml + 110 ml CMSL(6503/001)



Properties & Advantages :

- ▶▶ This is a 2 part rigid dental acrylic used for making dental appliances.
- ▶▶ Contains both powder and liquid components.
- ▶▶ Light Veined Pink Dental Acrylic stimulates the fleshy gum tissue of the mouth.
Indication for use : Used for full or partial upper or lower dentures

Description & Features

- ▶▶ Heat cured acrylic denture base material.
- ▶▶ Simple mixing, easy to process.
- ▶▶ Cross linking ensures superior quality.

Shelf life and storage : Best before 36 months under proper storage.
For Best results, Use with MAARC Heat Cure Liquid.

Description & Features:

- ▶▶ Working time of 4 minutes
- ▶▶ Rapid bench cure time of 14 minutes
- ▶▶ Also useful in orthodontic work

Prostho Range

Sticky Wax Sticks CE

Thickness : 114 x 8 mm dia sticks weighing appx. 6gms. / Pc

Packing	Product Code
10 pcs	2701/010
200 gms	2701/200

Properties & Advantages :

- ▶▶ It is blending of various waxes and resins and it is yellow in color.
- ▶▶ It melts when heated and adheres to the surface over which it is applied.
- ▶▶ It holds the components prior to fixation and gives a clean burnout.

Shelf life and storage : Best before 60 months under proper storage.



RESTORATIVE RANGE



ETC-37 CE



ETC-LIQUID CE



Known as : Etching Material- 37% Ortho Phosphoric Acid

Packing	Product Code
3 gm syringe + 3 Applicator tips	9401 / 003
25 gm syringe + 10 Applicator tips	9401 / 025
15 ml Liquid	9402 / 015

Benefits :

- ▶▶ Controlled Flow
- ▶▶ Precise Placement
- ▶▶ Selective Etching
- ▶▶ High Visibility
- ▶▶ Higher Resin Penetration, Hence Better Mechanical Retention

Description :

- ▶▶ Thixotropic gel
- ▶▶ Silicon free

Properties & Advantages:

- ▶▶ It is used in total etch technique for composite restorative, adhesives cementation of inlays, crowns, bridges, veneers, adhesive bridges and posts & cores.

Shelf life and storage :

Best before 24 months under proper storage.

Mercury



Packing	Product Code
30 gms	3001 / 030
225 gms	3001 / 225

Properties & Advantages :

- ▶▶ 99.99% Pure
- ▶▶ An especially triple distilled, chemically pure mercury.
- ▶▶ Recommended for preparing superior dental amalgam in combination with either of the Alloys in powder form.

Calcium Hydroxide Powder

Indication :

- ▶▶ Ideal for temporary root filling
- ▶▶ Pulp capping
- ▶▶ Pulp protection
- ▶▶ Provides insulation in deep cavities

Composition :


- ▶▶ Chemically pure calcium hydroxide in powder form

Properties & Advantages :

- ▶▶ Provides bactericidal effect on carious dentine.
- ▶▶ Paste delivered to root apex provides pulp insulation.
- ▶▶ Protects the vital pulp from infections and further decay.
- ▶▶ Can be easily removed from root canal by simple irrigation

Shelf life and storage : Best before 24 months under proper storage.

Packing	Product Code
15 gms	5002 / 015

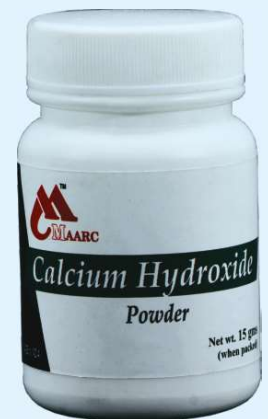

RESTORATIVE RANGE

ZIEGO

**MAARC
DENTAL**

Packing	Product Code
30 gm powder / 15 ml liquid	8701/001



The image shows the ZIEGO Zinc Oxide + Eugenol Kit. On the left is a white box with a purple and red design, featuring the MAARC DENTAL logo and the text 'ZIEGO Zinc Oxide + Eugenol Kit'. Below the box, it says '1 X Powder - 30gms, 1 X Liquid - 15ml' and 'MADE IN INDIA'. To the right of the box are two containers: a dark brown bottle labeled 'Eugenol 15ml' and a white jar labeled 'Zinc Oxide 30gms'.

Shelf life and storage :

Best before 24 months under proper storage.

Restorative Range

Eugenol

Packing	Product Code
30 ml	3002 / 030
110 ml	3002 / 110



Properties & Advantages :

- ▶▶ 98.9% pure.
- ▶▶ For temporary relief of localized dental pain when combine with zinc oxide to form a sedative dressing.
- ▶▶ Chemically pure with increased effectiveness.
- ▶▶ Forms a uniform paste consistency when mixed with zinc oxide.
- ▶▶ Forms a sedative dressing or lining under non composite resin when mixed with zinc oxide.

Shelf life and storage :

Best before 24 months under proper storage.

Zinc Oxide



Packing	Product Code
100 gms	5001/100

Properties & Advantages :

- ▶▶ 99.99% Pure
- ▶▶ Arsenic free, pure zinc oxide powder for temporary dental dressing.
- ▶▶ Used for restorative and prosthodontic applications.
- ▶▶ For best result use with MAARC Eugenol

Shelf life and storage :

Best before 60 months when stored in cool and dry place.

OTHER

Lubri Spray CE



Packing	Product Code
600 ml Can + Nozzle.	5506 / 600
150 ml Can + Nozzle.	5506 / 150

All purpose Hand piece Lubricant Spray for maintenance of all dental hand pieces including highspeed, slow speed and air motors.

Indications: Highly effective lubrication for all high and slow speed dental rotary hand pieces. Lubri Spray contains anti-rust agent that prevents micro rust on inner bearings. High penetrative ability to ensure thorough coating of bearing surface. Each can contains a high volume pressure to provide maximum bearing rotation during oiling. High internal pressure of 70+Psi (4.8atm) at 20°C. Cans are compatible with the most renowned brands. Clear in colour.

Directions for use: Insert into back of hand piece and spray through for one to two seconds immediately after each use and before sterilization. Wipe excess thoroughly. Important: Hold can in a vertical position when in use. Refer to hand piece maintenance instructions where applicable.

Shelf life and storage :
Best before 24 months under proper storage.

Other

Arch Bar CE



Packing	Product Code
Arch Bar 12 inch long	5508 / 012

Description: It is made of 316 grade SS suitable for rigid fixation, techniques in the dentate patient begin with fixation of the occlusion. This ensures that the patients maintain their preoperative occlusal status. There are several techniques to providing mandibulomaxillary fixation (MMF). Many surgeons agree that the gold standards in MMF is, the use of arch bars.

Indications: For temporary fragment stabilization in emergency cases before definitive treatment.

As a tension band in combination with rigid internal fixation,

- ▶▶ For long-term fixation in conservative treatment
- ▶▶ For fixation of avulsed teeth and alveolar crest fractures

OTHER



Glass Beads CE



A method used for sterilizing the working ends of endodontic files and reamers or small dental instrument tips by placing them in a container containing glass beads heated to approximately 225°C (437°F) for a defined period of time.

Packing	Product Code
Glass Beads 200 gms	5503 / 200

Relief Wax for Ortho CE



Packing	Product Code
(5 Strips)	2310 / 005

Properties & Advantages :

- ▶ Relief Wax consist of a medical grade silicone for temporary relief from appliance irritation.
- ▶ This patient relief wax lasts longer and is easier to use than ordinary orthodontic waxes.
- ▶ It is clear and waterproof and stays on the bracket longer for greater patient relief.

Shelf life and storage :

Best before 24 months under proper storage.

OTHER

Angular Alcohol / Spirit Lamp CE



Packing	Product Code
15 gms	5005/G/001

- ▶ Unique hexagonal shape
- ▶ With flame control and gas exit points
- ▶ Lamp can be used upright or tilted position to various angles for applying heat in right spots.


Other

Carding Wax / Teeth Wax CE

We manufacture two types of carding wax for Pouring and for Extrusion.

Properties & Advantages :

- ▶ Both the waxes are very stable at temperature from 12°C to 40°C.
- ▶ Pouring waxes are suitable for automatic machines for tray filling.
- ▶ Wax firmly holds the teeth and while teeth is pulled negligible amount of wax comes along with it.
- ▶ Teeth can be re-adhere gaining similar strength.

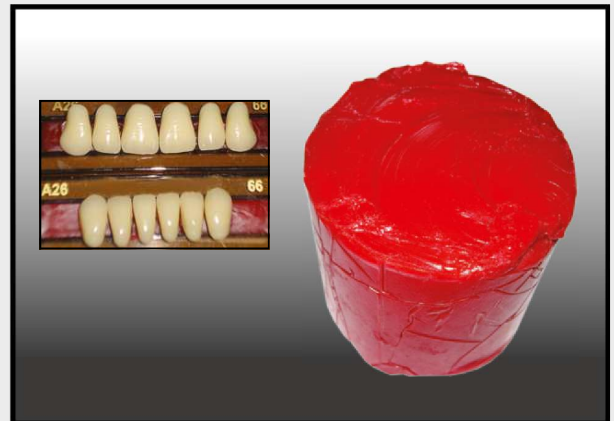

NOTES

Exhibitions 2020

Dates	Exhibitions	Stall no.
24-26 Jan	IDC Trivendrum	To be Announced
4-6 Feb	AEEDC Dubai	Hall 2, 210
8-9 Feb	Famdent Pune	B - 34
28-29 Mar	Expodent Chennai	To be Announced
24-26 Apr	IDEM Singapore	6Q-18, Level 6
29-31 May	Famdent Mumbai	To be Announced
5-6 Sept	Expodent Bengaluru	To be Announced
18-19 Sept	Expodent Chandigarh	To be Announced
3-4 Oct	Expodent Mumbai	To be Announced
25-27 Dec	Expodent Delhi	To be Announced



Shiva Products

ISO 13485 : 2016 (TUV SUD Certified)

Manufactured by :
Shiva Products

H.O.: 214, Bluebell Indl. Estate, Chinchpada, Vasai (East),
Dist. Palghar 401 208, Maharashtra, India.

☞: B-18, Dwarka Indl. Est., Naik Pada, Waliv, Vasai (East),
Dist. Palghar 401 208, Maharashtra, India.

Branch Office

Shop No. 7, Gola Market,
Pratap Street,
Nr. Golcha Cinema, Darya Ganj,
New Dehli-110 002. (INDIA)

SP | BR | 05 Rev. 05/13.01.2020

 : www.maarc dental.com |  : info@shivaproducts.net |  : +91-9987637290