

# FS-1



## Spring Adjustable Face Shield

Kit: 1 Headband

+10 Replacement Shields

- 180° Adjustable Ocular Shield
- Light-Weight for all day comfort
- Anti-Dust
- Anti-Droplet
- Anti-Splash
- Anti-Fog
- Detachable/Replaceable Shield
- Inside Distance from lens of regular glasses behind shield to inside of shield approximately 2"
- Length top to bottom 7.5" (190mm)
- Bottom curve of Shield 11.1" (282mm)




## Instructions for Use


PRODUCT STANDARDS: GB 14866-2006 / EN 166:2001 / ANSI/ISEA Z87.1D3

**Spring Health Products FS-1 Adjustable Face Shield. Face Shield is non-sterile disposable anti-fog Face Shield.**


The product and the protective packaging should be checked before using. Stop using it if there is any damage: Precautions: a. One-time use only per ocular shield. b. Use in accordance with the instructions. c. Never use it when damages are found. d. Stay away from chemicals e. The eye protector is not intended to protect against high speed particles. Storage: Please store in clean, dry, and ventilated indoor place, with relative low humidity.

 **Warning:** a. Materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals; b. Scratched, or damaged ocular Shields should be replaced; c. Eye-protectors against high speed particles worn over standard ophthalmic spectacles may transmit impacts, thus creating a hazard to the wearer. d. These protectors are intended for indoor use where no optical radiation hazards exist.

1. Fix the button (A) to hole (B).

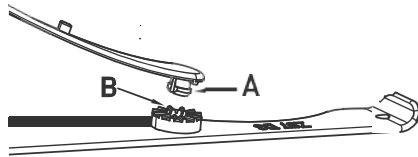

3. Adjust the strainer

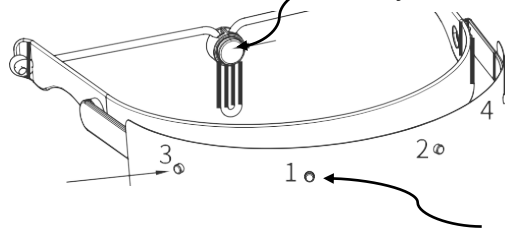

2. Please remove the protective membrane on both sides of the shield, then fix the shield on the frame by buckling up buttons 1-4 IN ORDER.

## ADDITIONAL SPRING HEALTH PRODUCTS



Diamond Burs



TC-3 Curing Light with the WIDE LENS and Low-Profile Head - USA



TC-24A Corded Curing Light with the WIDE LENS - USA



TC-CLII Cordless Curing Light Made in USA

